



Muting sounds on Intercall 600 and 700 Series

Technical Guide

v2.0



Blaucomm



This document will guide you through the muting the sounds for standard events (call, assistance, priority ect.) whilst leaving sound active for emergency events.

Note, this has to be completed on every display panel in the building.

Enter Menu Settings




Approach the display screen. Press and hold the 'Show Staff' button. While holding down the 'Show Staff' button, press & hold the 'Accept Call' button down. Hold both down together for 10 seconds.*

*Note that an L628 Display has a jumper which must be fitted to allow access to the menu, so if this doesn't work, please fit the jumper on the rear of the unit. If you're unsure how to do this, please do not proceed.

Also Please note while in the menu mode, the unit will not respond to call events.




Selecting SOUNDS FOR Menu

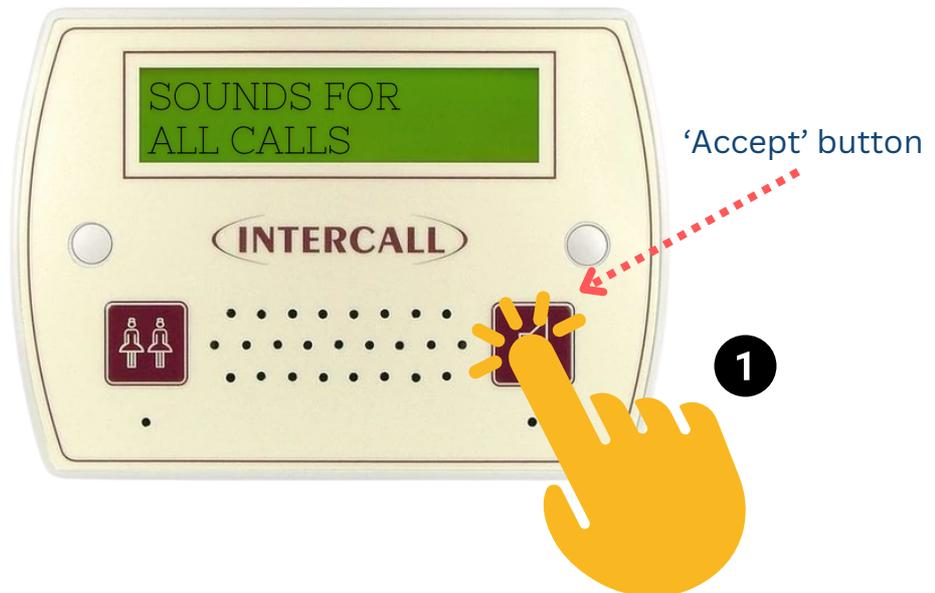


Use the 'Accept' button to scroll through the menu options until you see an option 'SOUNDS FOR'.

Changing SOUNDS FOR

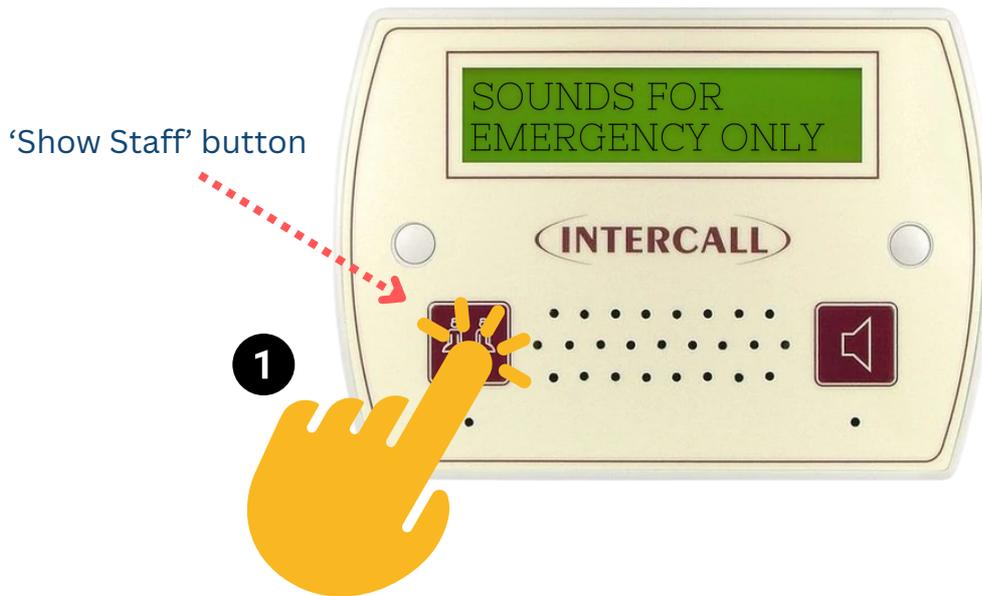

Press the 'Show Staff' button once to change the option from 'ALL CALLS' to 'EMERGENCY ONLY'.

Return to normal operation

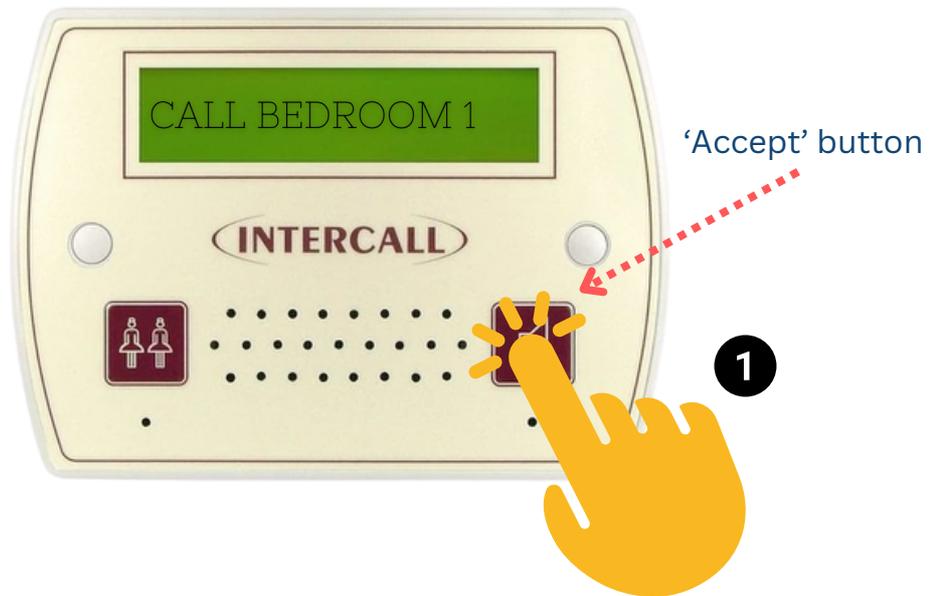

Now that you've applied the 'EMERGENCY ONLY' setting, use the 'Accept' button to scroll through the menu options until you return to the normal screen for alerts.

You're all done!

Great - that concludes how to physically mute the sounds for standard events but still receive emergency sounds on an Intercall 600 and 700 series display.

You can reach support via :

support@blaucomm.co.uk

03333 601 006

<https://blaucomm.co.uk/healthcare> and click "Live Chat"



Blaucomm