


# A guide to setting up your Silver Shield sensor after installation

## Seamless Protection with Smart Technology

The Silver Shield app is an essential companion for Silver Shield hardware, ensuring enhanced safety and wellbeing monitoring in homes and care settings.


### Step 1

Navigate to the 'Add Device' button.

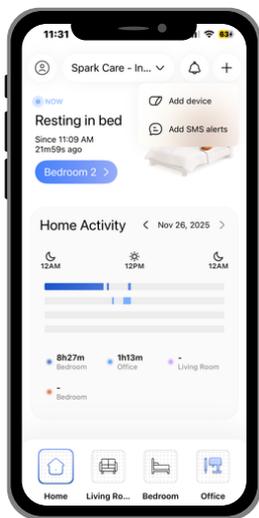

### Step 2

Familiarise yourself with the set-up process.

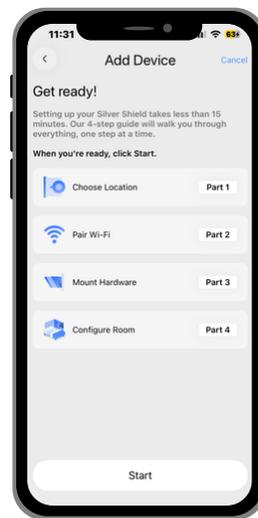

### Step 3

Select your room type.



 [spark-care.co.uk](https://spark-care.co.uk)



 01635 019608

 [hello@spark-care.co.uk](mailto:hello@spark-care.co.uk)

 The Old Granary, Harepath Farm, Burbage, Marlborough, Wiltshire, SN8 3BT


## Step 4

Read through the Device Installation Rules to select the best wall corner for installation.

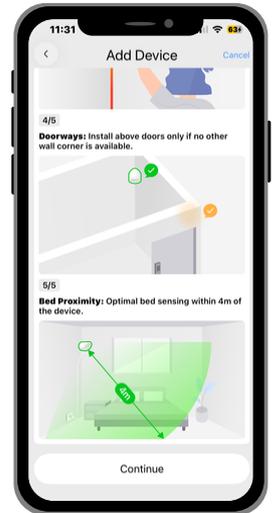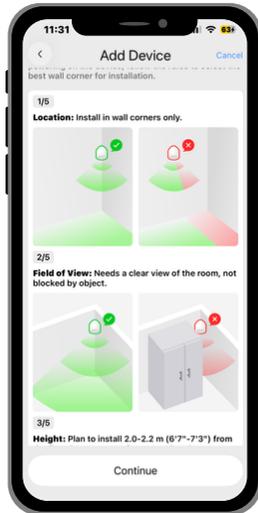


## Step 5

Once you have chosen the best corner to install the sensor, plug the adapter into the nearest power source and connect the cable to the sensor.

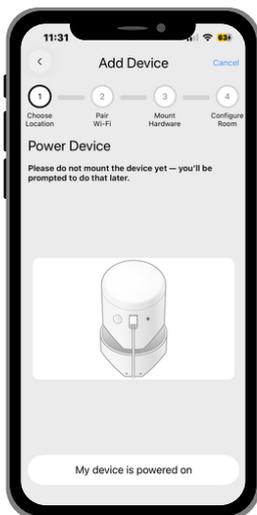



## Step 6

Ensure the sensor is in pairing mode, indicated by a green blinking light. Do not mount the Silver Shield sensor yet.

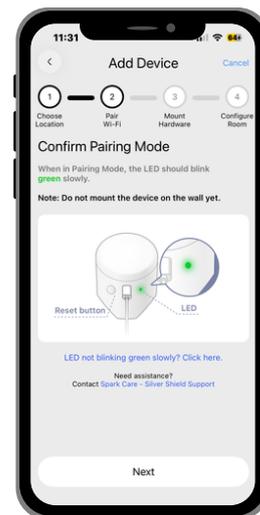

## Step 7

Select your Silver Shield device (indicated by the serial number on the box) and connect the device to your Wi-Fi network.

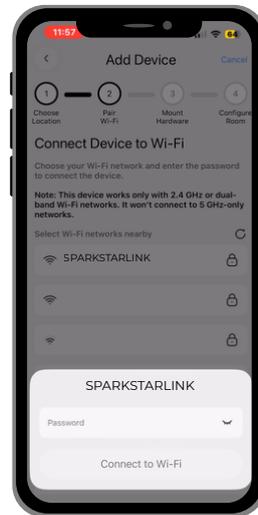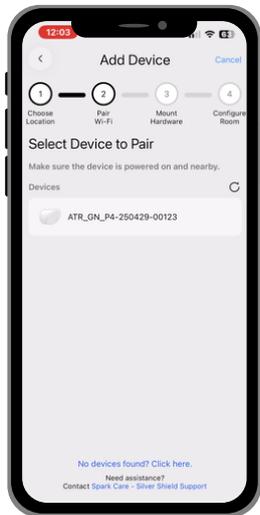





## Step 8

Select your mounting preference.



# Step 9

Follow the mounting guide to ensure optimal sensing performance of the Silver Shield device.

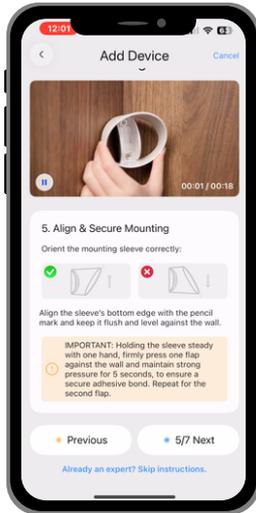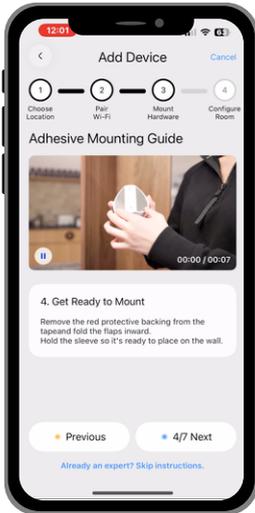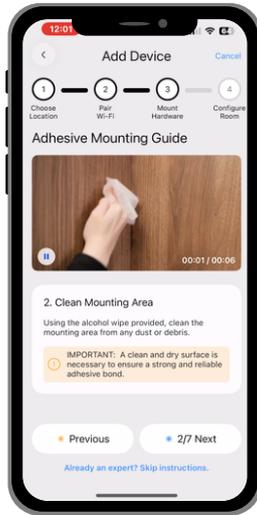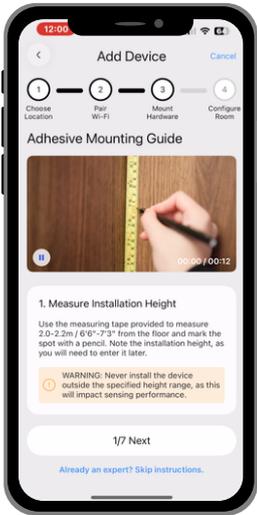




## Step 10

Double check room type, select and enter your room name.






## Step 11

Ensure correct height calibration for optimal performance of the Silver Shield. Ensure only one person is in the room for calibration. Calibrate the room size by starting underneath the Silver Shield, then slowly walk to each four corners of the room to capture the entire room. Ensure to double check the room dimensions with a tape measure so it's accurate.

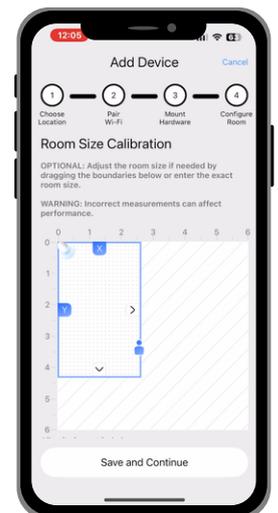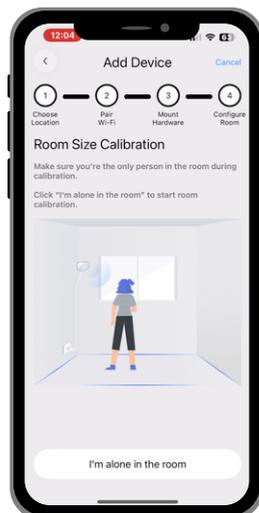
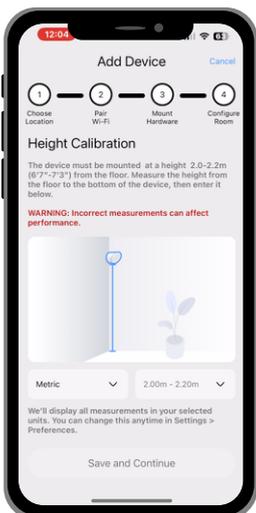


# Step 12

Zone the room and add elements such as the bed, chair, sofa, or toilet. Customising the room layout helps Silver Shield detect movement more accurately. Measure each element to ensure accuracy is maintained.

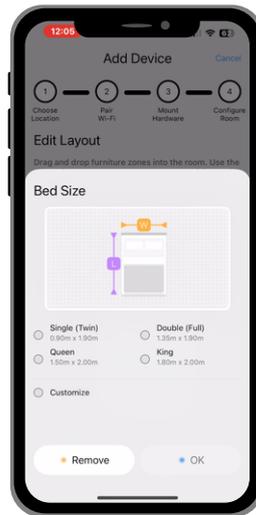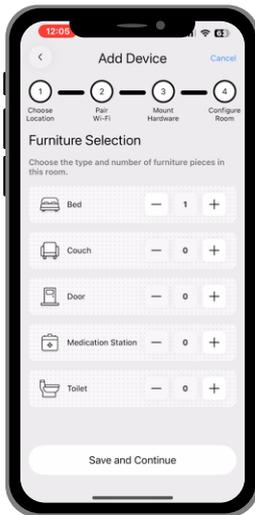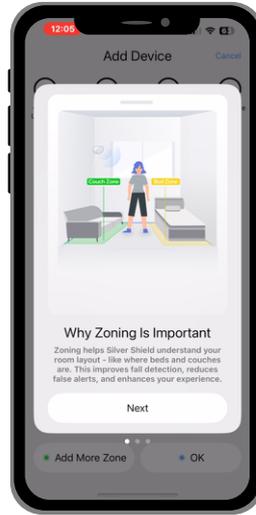


To make edits to zoning at a later point if the layout of a room changes, click the Settings icon on the room and scroll down to room calibration. Ensure to keep this up to date for optimal accuracy of the sensor.

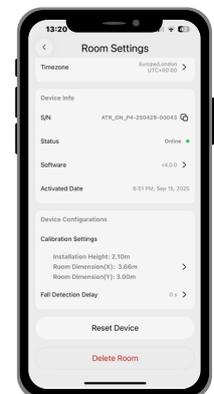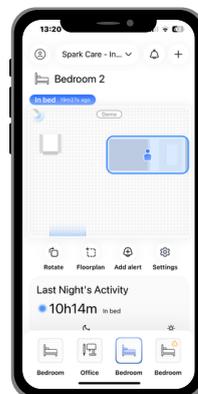

 spark-care.co.uk  
 01635 019608

 hello@spark-care.co.uk  
 The Old Granary, Harepath Farm, Burbage, Marlborough, Wiltshire, SN8 3BT

Your device set-up is now complete!  
Silver Shield is ready to keep you connected, informed, and in control.

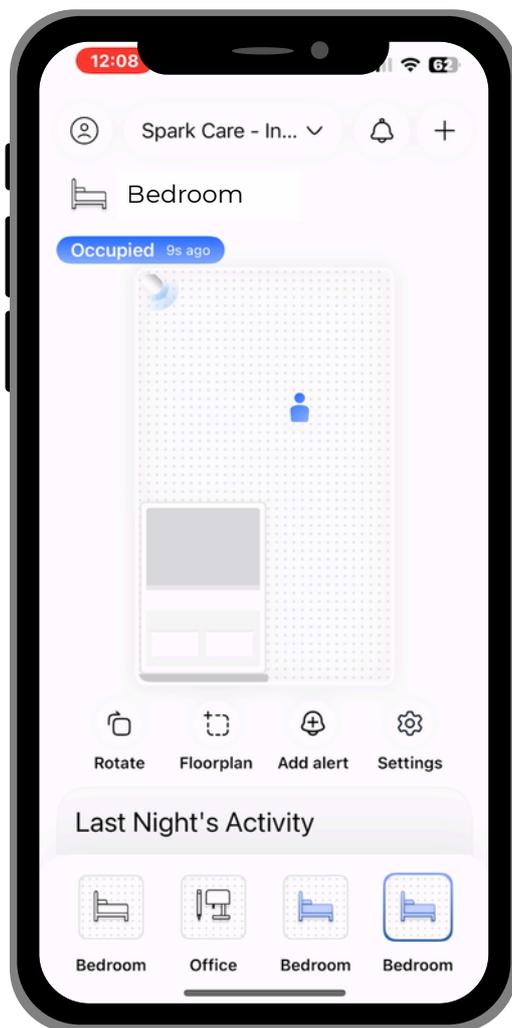

## Got anymore questions?

Contact Spark Care Customer Support either by phone, or email:



**01635 019608**



**cs@spark-care.co.uk**



spark-care.co.uk



01635 019608



hello@spark-care.co.uk



The Old Granary, Harepath Farm, Burbage,  
Marlborough, Wiltshire, SN8 3BT