

# Zoning Furniture



By Spark Care

Adding furniture zones helps define where key activities occur, such as lying in bed, sitting on the couch, or entering/exiting a room. This context allows caregivers to better understand behaviour patterns and identify high-risk areas for falls.

Available zone types:

- Bed
- Couch
- Door
- Medication station
- Toilet

**Required materials:** Measuring tape (only needed if using custom furniture dimensions)

**SILVER SHIELD**  
Knowledge Hub

**APP USER GUIDES**



 hello@spark-care.co.uk


 01635 019608

 spark-care.co.uk


 The Old Granary, Harepath Farm,  
Burbage, Marlborough, Wiltshire, SN8 3BT

# 1. Open the Room Layout


Navigate to the room you want to edit.

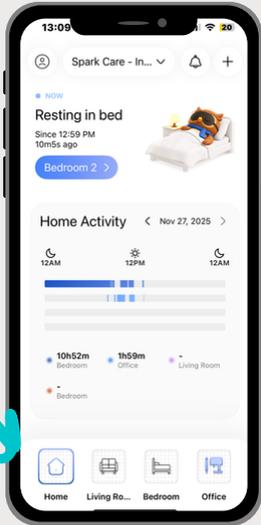

# 2. Press the Floorplan icon

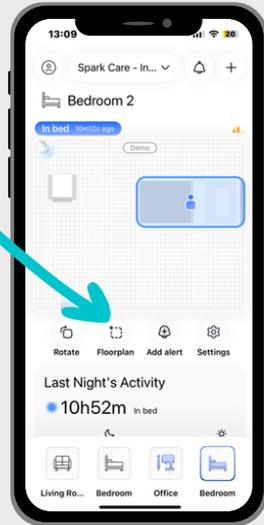


# 3. Press 'Add More Zone' button

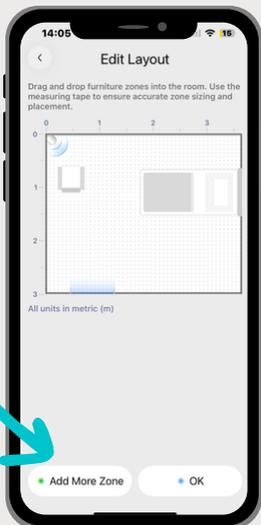


# 4. Select the Zone Type

Choose the furniture type you want to zone (e.g., bed, sofa).

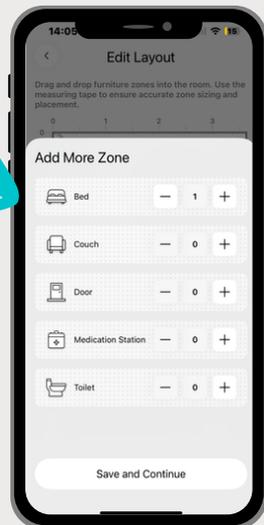


## 5. Select the furniture icon

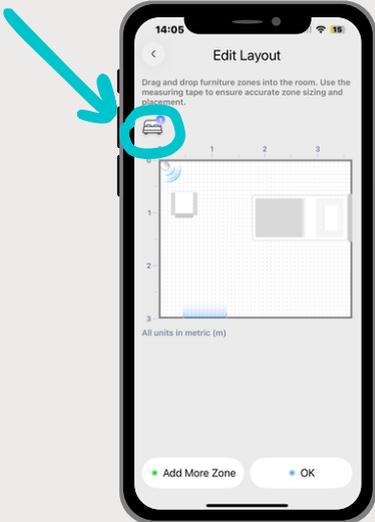

## 5. Choose a Size


Select a predefined size or choose Custom to enter exact dimensions.

- If Custom, measure the furniture using a measuring tape, and input the length and width.

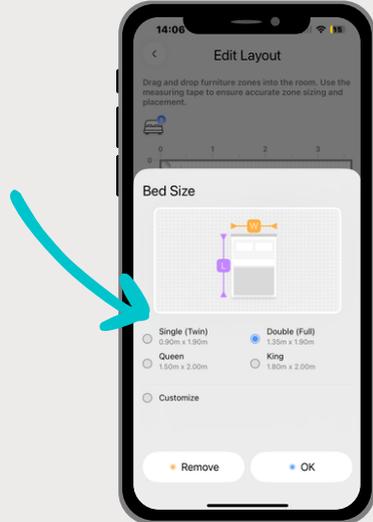

## 6. Position the Zone

Drag and place the furniture zone in the correct spot on the layout. You can also sit or lie on the furniture to see the presence icon move to help verify its position in real time.

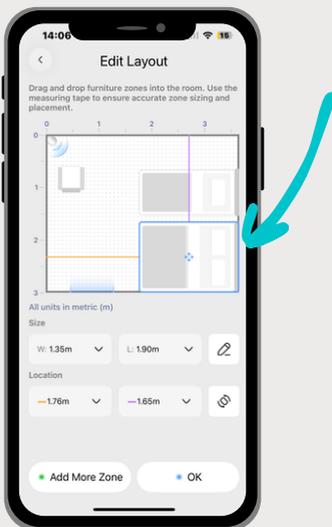

## 7. Save your Changes

Once the size and position are correct, tap OK.

